CAM BİBERON 250 ml

Glass Feeding
Bottle 8 oz

wee®
baby

D

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EN

WARNING FOR THE HEALTH AND SAFETY OF YOUR BABY!

•ALWAYS USE THIS PRODUCT UNDER THE SUPERVISION OF AN ADULT. •NEVER USE THE TEAT INSTEAD OF A REGULAR SOOTHER. •PROLONGED AND CONTINUOUS SUCKING OF LIQUIDS MIGHT CAUSE TOOTH CAVITY. •CHECK THE FOOD TEMPERATURE BEFORE YOU START FEEDING THE BABY. •KEEP ALL UNUSED PARTS OUT OF THE REACH OF CHILDREN.

Instruction For Use:

•Before each use pull the teat to all directions and examine carefully. Do not use if any damage or wearout is observed. •Hold in boiling water for five minutes before the first use. •This process is important to be sure that the product is impeccably clean. •Wash in warm and soapy water and rinse thoroughly before each use. •Do not leave in disinfectants for a long time. Don't expose to direct sunlight or direct heat sources, this may cause damage. •Store in a dry and closed container when not in use. •After using wash with a suitable brush until no food remains.

NOTE: •Do not use in a microwave. •Always stir hot food to make sure that the heat is equally dispersed. •Not suitable for cleaning in the dishwasher. •Suitable to clean in sterilizers.

IMPORTANT NOTE !

•Rinse the bottle with warm water before filling in hot food. •Do not use damaged bottles. •Glass bottles can break. •Sealing system of the PP Wide Neck PP Feeding Bottles and PP Mini Exercise Cups is located at the cap. Make sure that the soother nipple is fitted into the cap while closing after feeding. For activating the sealing systems of our other products, remove the feeding bottle nipple and place sealing washer inside the screw. Use the feeding bottle nipple during feeding only. •Do not leave any liquid inside if not used for a long time. Otherwise it may cause mould and odour.

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